



luxiva[®]

Skin Mesotherapy Solution

Rejuvenating Complex





Skin Aging

A Universal Concern

The skin is the corner stone of social trends at any age and in any culture

Women's Main Concerns

- Change in skin texture
- Wrinkles
- Dull complexion
- Sagging & reduced elasticity
- Pigmentation disorders



Flawless Skin Is Above All Healthy Skin

Skin that expresses positive, natural emotion reflects healthy elegant appearance and doesn't need additional make-up



Healthy Skin

- Radiant & even
- Hydrated
- Smoothed
- Tonic

(1) Skin quality study - WRC - 32 subjects aged 35 to 55 - May 2018.




Skin Aging

The Key Role of Fibroblasts


Fibroblast, a Major Cell in The Dermis

- Collagen production
- Elastin production
- Hyaluronic acid production
- Key role in maintaining youthful appearance of skin

Young Skin-Healthy Fibroblast

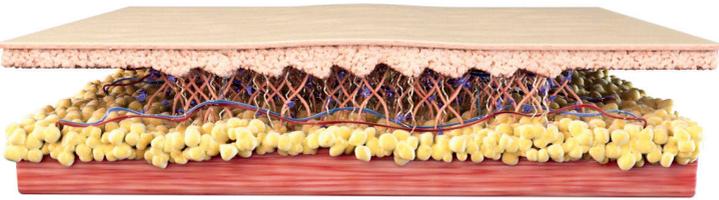

Mature Skin-Aging Fibroblast

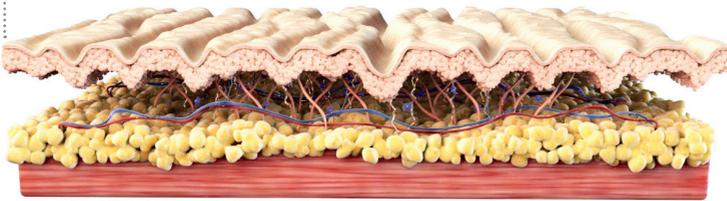

..... Aging
..... Oxidative Stress

- Reduction of quality and quantity of collagen - elastin - hyaluronic acid
- Imbalance between antioxidant protection and free radicals

Appearance of 1st Signs of Aging



luxiva® *Rejuvenating Complex*

Skin Mesotherapy Solution

Reboots Fibroblast Activity For a Reverse Aging Effect

4 vial 4 mL:

- **Hyaluronic acid** (20 mg/vial)
- **Vitamins:**
Thiamine, Riboflavin, Nicotinamide, Calcium Pantothenate, Pyridoxine, Biotin, Folic Acid, Cyanocobalamin
- **Minerals:**
Calcium Chloride, Potassium Chloride, Sodium Chloride, Magnesium Sulfate



- **Amino acids:**
Alanine, Arginine, Asparagine, Aspartic acid, Glutamic acid, Glycine, Histidine, Isoleucine, Leucine, Lysine, Methionine, Phenylalanine, Proline, Serin, Threonine, Tryptophan, Tyrosine, Valine

● **Sugar:** Inositol
Creates an Optimal Environment For Fibroblasts

A Complete & Unique Formula



Mode of Application

Maximise Ingredient Penetration:

A four-step protocol comprising multiple micro-injections into the dermal-epidermal junction or the superficial dermis with 26-32G needles, is recommended for optimal and lasting results.

Penetration Devices



syringe



needle pen




roller



meso gun

Multiple Injection Techniques

● ● ● Point per point

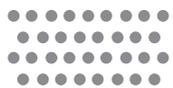

Nappage



Retrotracing across the wrinkle

Before

After



Before Week 12 Week 16



Before Week 12 Week 16



Before Week 12 Week 16



Before Week 12 Week 16



Before Week 12 Week 16



- 📍 Biotech Center, NIGEB
Tehran, Iran
- ☎ +98-942 600 1516
- ✉ info@atrazistaray.co.ir
- 🌐 atrazistaray.co.ir
- 📷 atrabiox
- 📍 atra.zist.aray