



Italian Formulation

NUXIWELL

**The New Generation
of Dermal Fillers**



NUXIWELL

Quality + Safety + Wise Choice



The New Generation of Hyaluronic Acid Dermal Fillers for Soft Touch and Natural-looking Effects

Nuxiwell fillers are the result of a collaboration between Italy and Iran, using the highest hyaluronic acid quality, as the main ingredient. Hyaluronic acid is a natural substance in the skin that helps to keep it young and fresh.

Nuxiwell products use cross-linked hyaluronic acid of different molecular weights to enhance their effectiveness.

Nuxiwell fillers come in four types of cross-linked hyaluronic acid with BDDE. These products can last from 6 to 12 months.

Each Nuxiwell box contains 2 ready-to-use 1 cc syringes and 2 sterile needles.

NUXIWELL 16



Nuxiwell 16 is a cross-linked hyaluronic acid gel that is sterile, pyrogen-free and compatible with the body's pH.

It does not contain any animal-derived components.

HA Concentration: 16 mg/ml **Volume:** 2x1.0 ml **Needle Size:** 30G^{1/2}

Injection depth: Mid Dermis

Injection Areas: Under Eye

Indication: Replace tear trough lost volume, improving the under eye shadow.

Complex Viscosity(G'): 82 Pa.s **Crosslinking Level:** ●○○○

NUXIWELL 20



Nuxiwell 20 is used to correct fine lines and wrinkles.

Nuxiwell 20 has a good balance of firmness, adhesion, viscosity and duration of effect 8 to 12 months.

HA Concentration: 25 mg/ml **Volume:** 2x1.0 ml **Needle Size:** 27G^{1/2}-29G^{1/2}

Injection depth: Mid Dermis

Injection Areas: Fine lines, Forehead lines, Glabella, Crow's feet

Indication: Correction Superficial Lines

Complex Viscosity(G'): 85.7 Pa.s **Crosslinking Level:** ●●○○

NUXIWELL 25



Nuxiwell 25 is suitable for restoring lost volume and removing deep wrinkles. The goal is to make the skin more even and smooth, and to create a fresher and younger look. Duration of effect 8 to 12 months.

HA Concentration: 25 mg/ml **Volume:** 2x1.0 ml **Needle Size:** 27G^{1/2}

Injection depth: Mid/Deep dermis

Injection Areas: Forehead, Naso-labia Folds, Temples, Correction of the lip corners

Indication: Correction Moderate to Severe Lines

Complex Viscosity(G'): 324.2 Pa.s **Crosslinking Level:** ●●●○

NUXIWELL 30



Nuxiwell 30 is suitable for adding volume to the face and enhancing or plumping the lips. Nuxiwell 30 can last from 8 to 12 months, depending on various factors such as a person's age and metabolism rate.

HA Concentration: 25 mg/ml **Volume:** 2x1.0 ml **Needle Size:** 27G^{1/2}-25G^{1/2}

Injection depth: Subcutaneous

Injection Areas: Lips, Mid-face volume

Indication: Shape and Restoration of Facial Volume


Complex Viscosity(G'): 491.3 Pa.s **Crosslinking Level:** ●●●●

Process



Maximization of Cross-linking Efficiency

Nuxiwell through the improved physical cross-link process to achieve a complete cross-linking level, while minimizing the amount of BDDE as a chemical catalyst material, which leads to manufacturing a dermal filler with high viscoelastic properties.

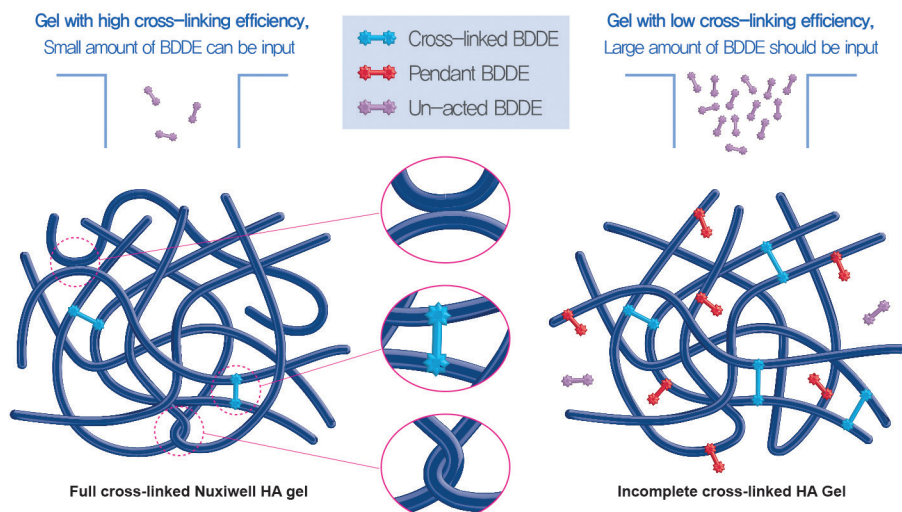



Pre-injection care

- At least 10 days before the dermal filler injection, avoid taking aspirin, anti-inflammatory drugs (such as ibuprofen, etc.), fish oil, supplements containing ginkgo, garlic, ginseng, vitamin E, and any other vitamin or drug that can increase bleeding tendency.
- Do not drink alcohol at least 24 hours before injection day. Alcohol can increase bleeding and bruising.
- Avoid using topical creams such as tretinoin, retinol, retinoid, glycolic acid, or any other topical anti-aging product 2 days before injection.
- On the day of the treatment session, come to the clinic with a clean face. Do not use cosmetics and moisturizing creams.
- Eat a light meal before injection appointment.
- If you have a history of herpes (herpes simplex virus), be sure to tell your doctor in advance.
- **Storage:** 2 - 25°C (Do not freeze).



Post-injection care

- Apply ice compress immediately after injection (continue 24 to 48 hours after injection).
- Do not massage or press the injected area as it can cause the filler to move.
- Do not use cosmetics for 24 hours.
- For the first 3 nights, sleep on your back and use 2 pillows to keep the dermal filler in place.
- Avoid heavy activities and sports for 48 hours.
- Bathing is not a problem, but avoid putting pressure and massaging the injected area.
- Avoid going to the swimming pool and jacuzzi and sunbathing for 1 week after the injection.
- It is important to drink enough water, as the dermal filler needs enough water in the body to achieve the best results. Lack of water in the body can affect the performance of dermal fillers.
- To help with bruising, we recommend using medications containing bromelain such as PINAHEAL capsules.
- In lip segment injections, avoid consuming hot foods and drinking hot liquids for about 1 week after dermal filler injection. Eating hot food can reduce the longevity of the filler.
- Avoid smoking and alcohol for at least 1 week after injection. These substances can increase the risk of swelling and inflammation.

Nuxiwell 16



Nuxiwell 20



Nuxiwell 25



Nuxiwell 30





 www.amitismed.com

 021-91304900

 amitismed

